

Maintenance and Lubrication of Propphy Angle / Contra Angle

J1180D3- Propphy Angle

After every polishing dip head of propphy angle into oil and run for short amount of time

Approximately after every 10 dental polishing perform propphy angle maintenance below



1. Remove propphy angle from handpiece
2. Remove head gears by turning knurled knob clockwise (threads are reversed) turn to your right.
3. Clean with fine brush and lubricate head gears with mineral oil, Vaseline or any non-toxic lubricant.
4. Place head gears back and tighten by turning Left or counterclockwise.
5. Normal polishing speeds are around 2,000 to 3,000 RPM, exceeding this may damage your propphy angle.



J1180D5- Contra Angle- Perform Lubrication after every use.



Lubrication Points

1. Remove Contra angle from handpiece
2. Locate lubrication points on back of contra angle
3. Lubricate with oil pen or by dropping oil into holes and slowly letting oil drain into contra angle.